TK-Soft mini

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CE 0044

TK-Soft mini repair element

TK-Soft mini as a precision-adjustable repair element for subsequent installation in telescopic crowns with low friction.

In the case of telescopic crowns whose friction has decreased, it is also possible to install the TK-Soft mini at a later stage. When positioning the friction element, a sufficiently stable connection between the secondary telescope and the denture saddle is a prerequisite. The corresponding receiving channel for the friction element can be created using standard carbide cutters. If this space is not available, the repair elements Quick-tec or Quick-tec plus should be used as an alternative.

1. Processing technique

►1 Initial situation: The friction-free telescopic denture.

►2 Create a working die made of putty silicone. This is shortened down to the crown margin in the area where the TK-Soft mini is to be inserted later.

►3 At the selected position for the friction element, usually in the direction of the denture saddle, a receiving channel is milled according to the size of the TK-Soft mini..

►4 The milling of the receiving channel is carried out with the round head milling cutter. Subsequently, the final shaping takes place with the cylindrical milling cutter..

►5 The dimensioning of the receiving channel can easily be checked with the placeholder, positioned on the placeholder remover. The element must be located in the parallel area. A positioning that is too deep in the area of a chamfer will later cause the denture to be levered out of the end position.

►6 Now glue the TK-Soft mini placeholder with superglue above the flute at the corresponding position on the silicone die. The opening for the placeholder remover should point basally.

►7 Si-tec adhesive is used for the secure and long-term stable receiving of the TK-Soft mini element. The receiving channel is sandblasted, cleaned, and filled with a sufficient amount of adhesive to form the exact receiving cavity for the element. Now insert the silicone die with the placeholder into the secondary crown. After the adhesive has cured, pull the silicone die out of the secondary crown.

►8 If there is excess adhesive in the basal area of the placeholder, remove it and pull out the placeholder using the placeholder remover.

The TK-Soft mini element is inserted into the receiving channel formed by the adhesive using the mounting aid. The friction is adjusted individually later during re-insertion.

In the case of the TK-Soft mini, the narrow side of the placeholder must be fixed to the silicone die..

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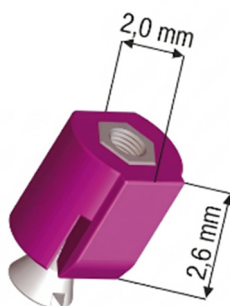
- TK-Soft mini starter set

2. Intended purpose

The TK-Soft mini manufactured by Si-tec is a repair element for telescopic crown technique.

The TK-Soft mini repair element is capable of restoring lost holding forces via friction by projecting into the crown interior of the secondary telescope. The friction can be individually adjusted by turning the activating screw and can also be readjusted later. The system is used for retrospective integration into telescopic crowns. It is inserted into a receiving channel milled with an abrasive tool and formed from cured adhesive.

3. Indication

Use as a repair element for parallel-milled telescopic crowns which have insufficient holding force.

4. Contraindication

TK-Soft mini is not suitable for:

- conically milled primary parts.
- patients who, for health reasons, are unable to comply with the regularly required check-up appointments.

In patients with allergies or suspected allergies to the materials contained in the product or the adhesive used, TK-Soft mini must not be used.

5. Expected clinical benefit

restoration of the holding force of a telescopic prosthesis that improves the oral health and quality of life of patients.

6. Target patient group

Patients with residual dentition and completed bone growth for whom fully fixed dentures are not possible or indicated.

7. Intended users

Healthcare professionals (dentists, dental technicians).

The dentist is responsible for explaining to the patients the correct handling during insertion and removal of the denture as well as the correct cleaning.

8. Qualification of healthcare professionals

It is assumed that professional knowledge in professional dentistry or dental technology is available. The current instructions for use must be available and observed. The production and regular check of dentures may only be carried out by qualified professionals.

9. Storage

Keep away from sunlight, in the original packaging.

10. Materials

Hostaform MT8U01, Titanium Grade 4

11. Side effects / Warnings

Side effects are excluded if used properly.

12. Single use

The product is intended for single-use. The product cannot be re-used because it is firmly connected to the denture and removal leads to a lack of function or destruction of the products.

13. Joint use with other products

For secure fixation of the product in prosthetics, only CE-certified attachment composites (adhesives) approved for the dental sector may be used. According to the manufacturer's instructions, the adhesive used must be explicitly indicated for bonding with the present materials (see point 10). The respective processing and safety instructions of the adhesive manufacturer must be strictly observed.

14. Sterility status

The products are delivered non-sterile.

15. Cleaning / Disinfection

VAH-listed, material-compatible cleaning agents and disinfectants may be used for cleaning and disinfecting the medical device. The following processes are suitable:

- Immersion bath with suitable cleaning agents and disinfectants (e.g. based on quaternary ammonium compounds, amines or peroxides)
- alcoholic immersion bath and spray disinfection ($\approx 70\text{--}80\%$ ethanol/propanol)

The concentrations and exposure times specified by the respective manufacturer must always be observed.

Residues must be removed after cleaning and disinfection with running water of drinking water quality.

16. Reporting of incidents

Serious incidents in connection with the product must be reported by the user to the manufacturer and to the competent authority of the Member State in accordance with the applicable legal requirements.

17. Symbols and their meaning

	Manufacturer		Consult instructions for use
	Date of manufacture		Medical Device
	Batch code		Unique Device Identifier
	Catalogue number		Health Industry Bar Code Standard
	Do not re-use		CE marking
	Keep away from sunlight		

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info@si-tec.de

www.si-tec.de

Certified Quality Management System
EN ISO 13485

CE 0044

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Si-tec GmbH